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1 — ⚠ **ADVERTENCIA CRÍTICA** (Lea antes de usar)

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2 — Resumen · Resumen del Sistema

VR WiFi setup is already completed.

Just turn on the machine and you can access
our application directly — no additional setup needed.

WiFi pre-configured

Ready to use

The access instructions below are provided for reference in case something goes wrong. Please keep them handy.

2 — Resumen · Resumen del Sistema

How to Perform the VR Perimeter Test

Follow the steps below to ensure accurate testing.

1 Prepare the System

WiFi Router

Mini PC (Server)

- ✓ Make sure the WiFi router and Mini PC are powered on.
- ✓ Confirm your device is connected to the **VRPerimeterG** network.

2 Wear the VR Headset

Put on the VR headset and adjust the head strap for a comfortable and stable fit.

3 Check Connection (Waiting Screen)

After wearing the headset, you will see the waiting screen in the VR world.

Green = Connected

Connection is normal. You can proceed to start the test.

Red = Connection Issue

Connection issue. Please check the WiFi router, Mini PC, and network (**VRPerimeterG**).

4 Start the Test

On the clinician's device, start the test. Follow the on-screen instructions.

5 Complete the Test

When the test is complete, results will be displayed automatically.

Tips for Accurate Results

- Keep your head still and your gaze on the center point.
- Avoid looking around or blinking excessively.
- Ensure the room is appropriately lit and free from distractions.

Connection Issue? (Shown in the VR World)

If the indicator below the woman's eye is red on the waiting screen, there is a connection issue. Check the power of the WiFi router and Mini PC, and make sure your device is connected to the **VRPerimeterG** network.

Green = Connected

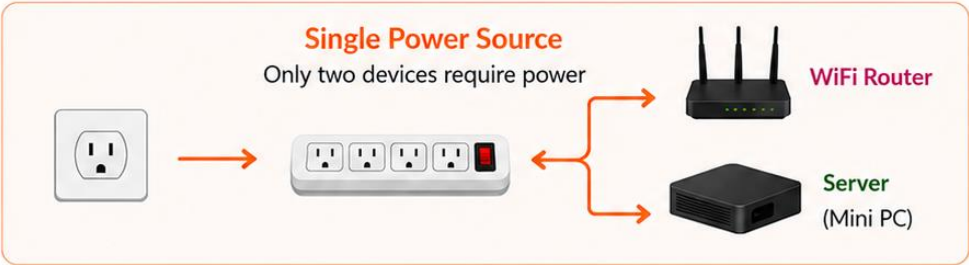
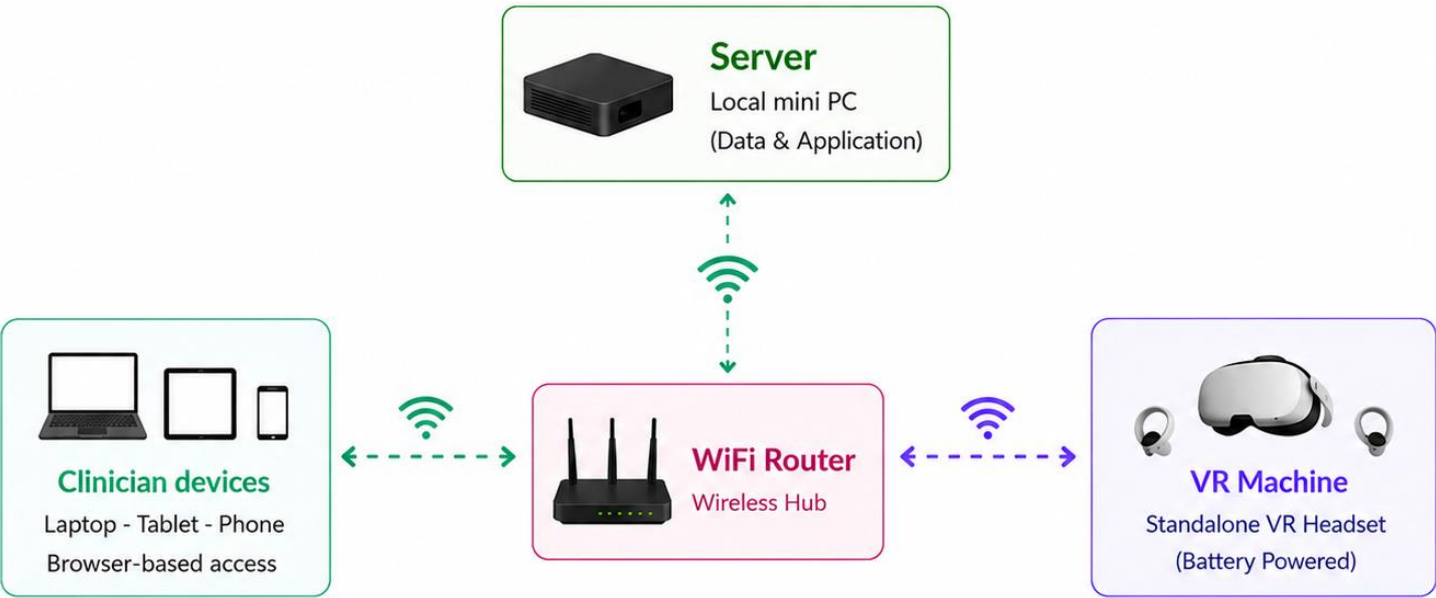
Red = Connection Issue

2 — Resumen · Resumen del Sistema

Isolated Local Network (All wireless, no internet connection)

← --- → Wireless (bidirectional)
→ Power supply

- ✓ No internet connection
- ✓ All communication stays within local network
- ✓ Only Server and Router require power

⚡ Only one power outlet required for the entire system

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You've left the Play Boundary

Please return or set up a new boundary.

Reset Stationary Boundary

Custom Boundary

Set Up Stationary Boundary

Small

Medium

Large

Standing

Sitting

Before entering VR, look around to make sure the boundary is correctly positioned and the area is clear of any obstructions that you might hit with your arms outstretched.

For more settings, go to Settings > Settings > My Settings > VR Settings.

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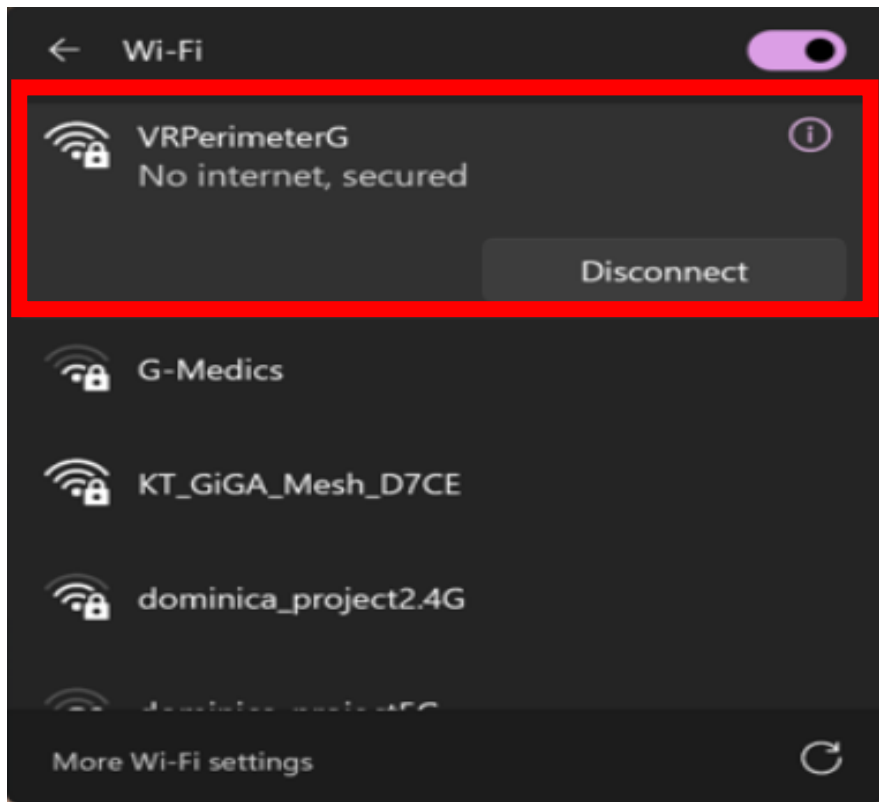
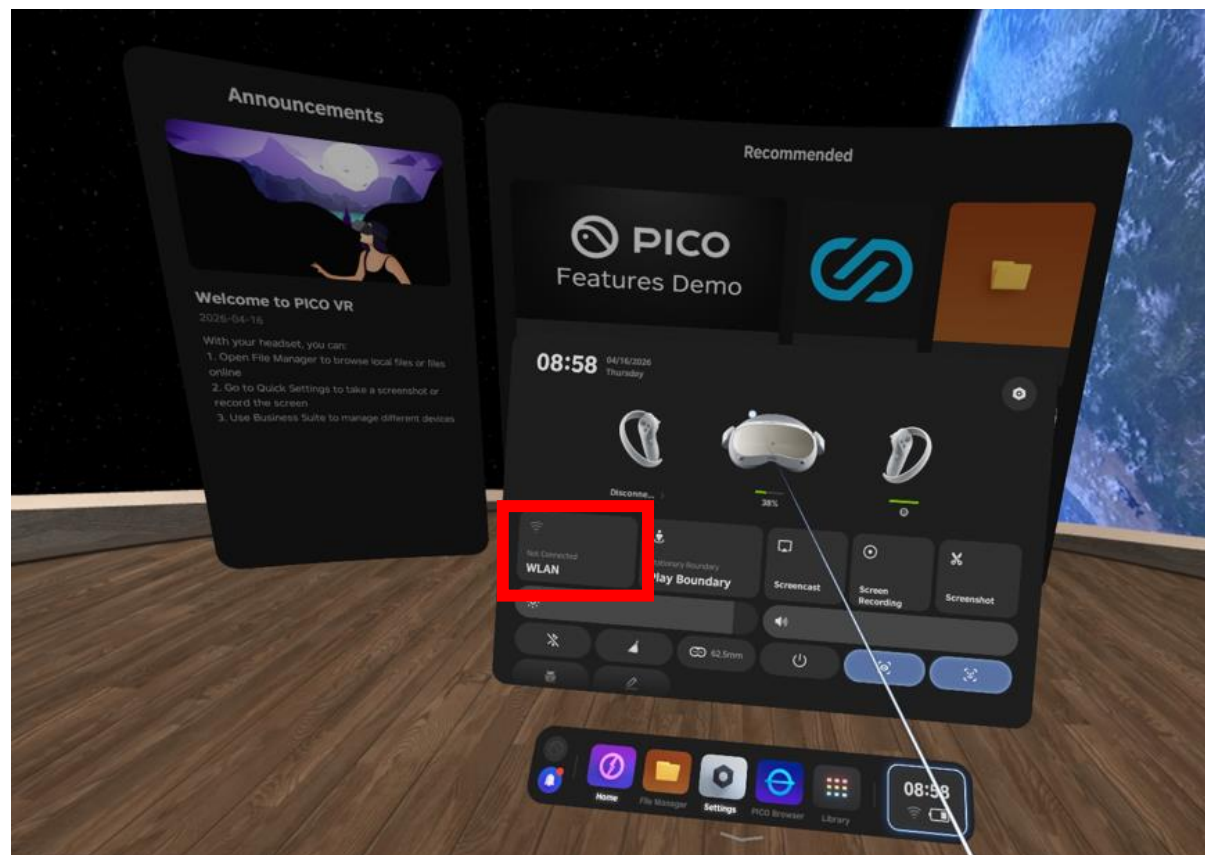
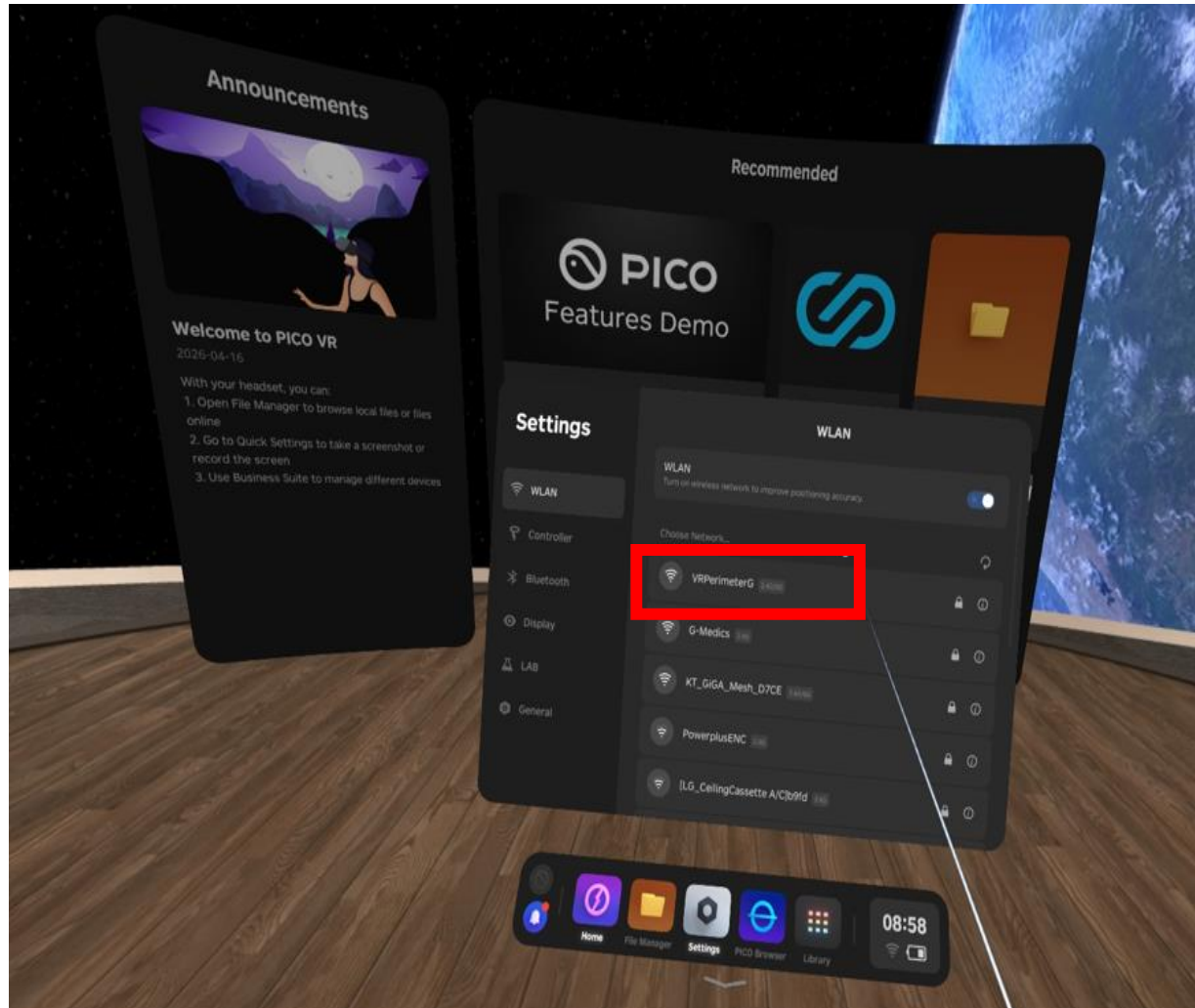
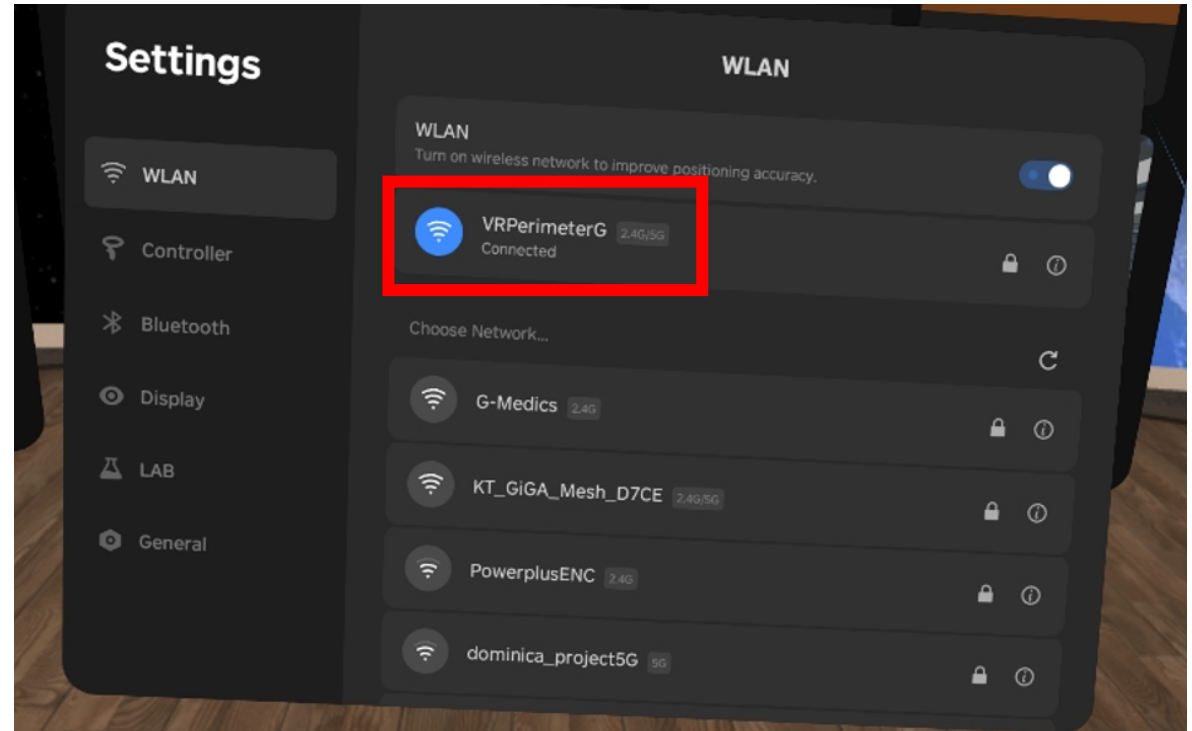
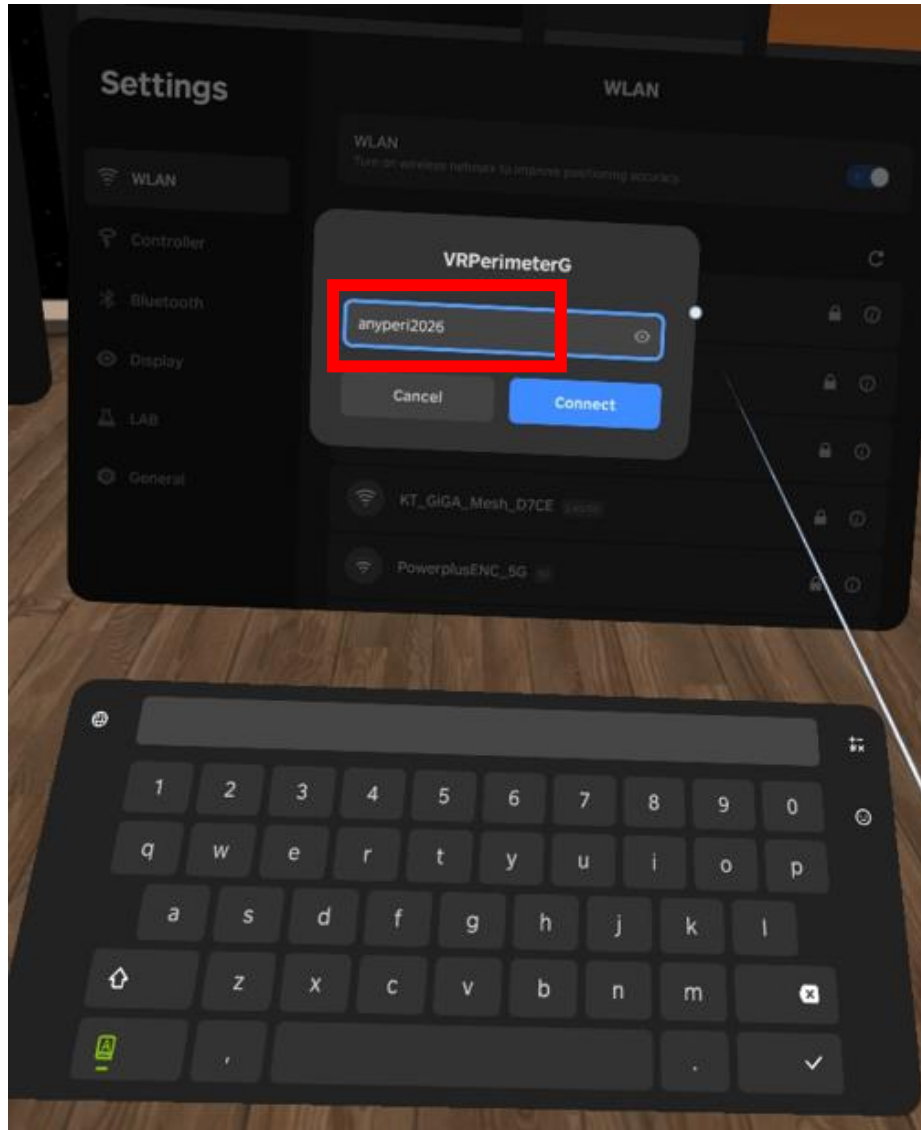
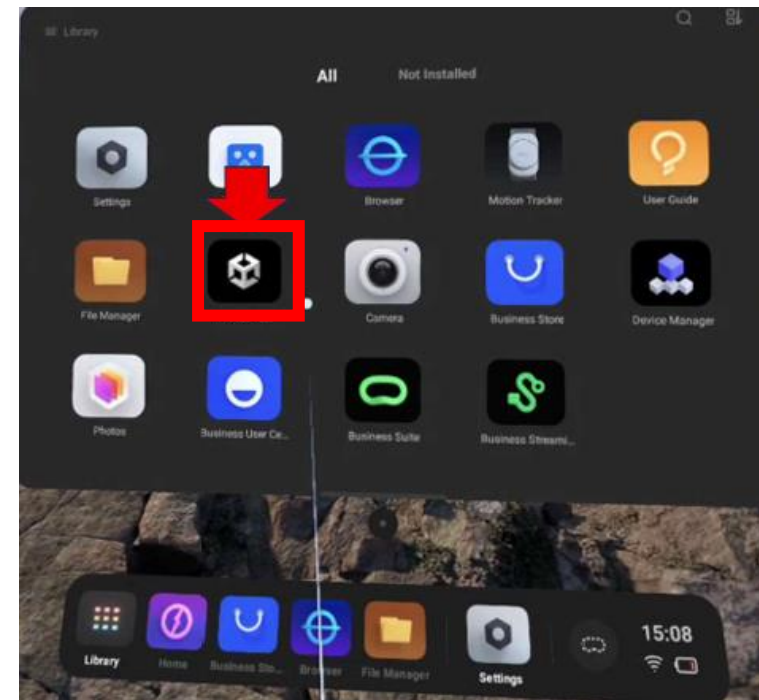
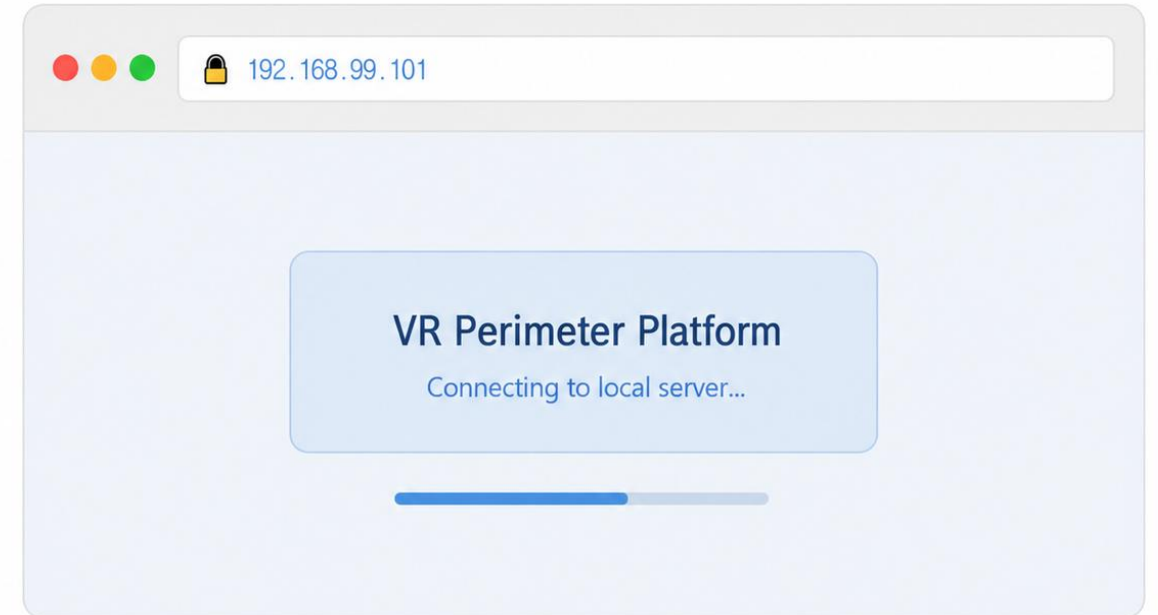
Login

HOW TO ACCESS — OPEN YOUR BROWSER

- 1 Make sure your device is connected to the WiFi network VRPerimeterG
- 2 Open any browser (Chrome, Safari, Edge) and type 192.168.99.101 in the address bar
- 3 Log in with your credentials — ID admin@hospital.com PW admin

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User Manual

 System Language Settings

Select the system language. Changes will be applied immediately.

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 Add New Patient

Patient List

NO.	PATIENT NAME	PATIENT ID	GENDER	DATE OF BIRTH	AGE	LAST TEST	TEST DATE	TEST STATUS	ACTIONS
1	tom	p0000131	Male	1993-06-08	33years	VFT-24-2	2026-06-23 13:14	Completed	    
2	이상훈	32332	Male	1973-10-13	52years	VT	2026-06-23 18:35	Completed	    
3	유채원	86139	Female	1998-11-26	27years	VFT-SUP-64	2026-04-21 12:09	Completed	    
4	Dennis	p000019	Male	1997-04-16	29years	VFT-24-2	2026-05-01 21:50	Completed	    
5	Bob	p000002	Male	1970-07-01	55years	CBT	2026-06-24 06:25	Pending	    
6	Alice	p000001	Female	2001-02-16	25years	VFT-24-2	2026-06-20 04:25	Cancelled	    

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Add New Patient

Patient List

NO.	PATIENT NAME	PATIENT ID	DATE	TEST STATUS	ACTIONS
1	이상훈	32332	2020-04-26	Cancelled	    
2	유채원	86139	2021-12-09	Completed	    
3	Dennis	P000019	2021-11-20 21:50	Completed	    
4	Bob	P000002	2020-02-25 02:35	Cancelled	    
5	Alice	P000001	2026-06-20 04:25	Cancelled	    

1-5 of 5

 Add New Patient

Patient Name *

Enter patient name

Patient ID *

Enter patient ID

Gender *

Select gender

Date of Birth *

YYYY-MM-DD

Notes

Enter notes (optional)

Cancel

Save

June 2026

Sun	Mon	Tue	Wed	Thu	Fri	Sat
31	1	2	3	4	5	6
7	8	9	10	11	12	13
14	15	16	17	18	19	20
21	22	23	24	25	26	27
28	29	30	1	2	3	4
5	6	7	8	9	10	11

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5 — Configuración del Software

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 AnyPeri

 Patient Management

 Test Management

 Report Management

 Settings

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Test Management

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116
Total Tests

0
In Progress

0
Pending

96
Completed

All

All

Refresh

Add New Test

Test Management

NO.	TEST TYPE	STRATEGY	TEST EYE	TEST NAME	PATIENT NAME	STATUS	PROGRESS	DATE / TIME	ACTIONS
1	VT	Landolt C	Both Eyes	VT Retest	이상훈	Cancelled	44%	2026-06-20 04:26	   
2	VFT-24-2	Fast Strategy	Right Eye	VFT-24-2 Retest	Alice	Cancelled	1%	2026-06-20 04:25	   
3	VFT-24-2	Fast Strategy	Right Eye	VFT-24-2 Retest	Alice	Cancelled	5%	2026-06-20 04:25	   
4	VFT-24-2	Fast Strategy	Right Eye	VFT-24-2 Retest	Alice	Cancelled	0%	2026-06-20 03:12	   
5	VFT-24-2	Fast Strategy	Right Eye	Visual Field Test	Alice	Cancelled	0%	2026-06-20 02:38	   
6	VT	Landolt C	Both Eyes	VT Retest	이상훈	Cancelled	0%	2026-06-20 02:37	   
7	CBT	Ishihara	Right Eye	CBT Retest	Bob	Cancelled	0%	2026-06-20 02:35	   
8	CBT	Ishihara	Right Eye	CBT Retest	Bob	Cancelled	0%	2026-06-20 02:32	   

5 — Configuración del Software

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AnyPeri Test Management

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+ Add New Test

Select test type

Visual Field Test
Visual Field Test

Visual Acuity Test
Vision Test

Color Blindness Test
Color Blindness Test

Select Patient

Search: b

Bob
P000002 | 1970-07-01 | M

Test Name *

Visual Field Test

Memo

Enter notes (optional)

Cancel **Add New Test**

5 — Configuración del Software

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The screenshot displays the 'AnyPeri' software interface. On the left is a dark sidebar with navigation icons and labels: 'AnyPeri', 'Patient Management', 'Test Management' (highlighted), 'Report Management', and 'Settings'. The main area shows the 'Test Management' page with a header 'Test Management' and a user profile '홍길동 한국대병원'. A modal window titled 'Add New Test' is open in the center. It features a large text area at the top for a title. Below it, under 'Test Settings', are several dropdown menus: 'Test Eye' (Right Eye), 'Test Dot Size' (III), 'Test Pattern' (24-2), 'Test Strategy' (Fast Strategy), 'Instruction' (On), 'Use Binocular Fixation' (Yes), 'Foveal Threshold' (Yes), 'Stimulus Speed' (Slow), and 'Fixation Position Design' (Standard). At the bottom right of the modal are 'Cancel' and 'Add New Test' buttons. The background shows a list of test results with columns for status, refresh, and delete icons.

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Add New Test

Test Management

NO.	TEST TYPE	STRATEGY	TEST EYE	TEST NAME	PATIENT NAME	STATUS	PROGRESS	DATE / TIME	ACTIONS
1	VFT-24-2	Fast Strategy	Right Eye	Visual Field Test	Bob	Pending	0%	2026-06-22 15:51	   
2	VT	Landolt C	Both Eyes	VT Retest	이상훈	Cancelled	44%	2026-06-20 04:26	   
3	VFT-24-2	Fast Strategy	Right Eye	VFT-24-2 Retest	Alice	Cancelled	1%	2026-06-20 04:25	   
4	VFT-24-2	Fast Strategy	Right Eye	VFT-24-2 Retest	Alice	Cancelled	5%	2026-06-20 04:25	   
5	VFT-24-2	Fast Strategy	Right Eye	VFT-24-2 Retest	Alice	Cancelled	0%	2026-06-20 03:12	   
6	VFT-24-2	Fast Strategy	Right Eye	Visual Field Test	Alice	Cancelled	0%	2026-06-20 02:38	   
7	VT	Landolt C	Both Eyes	VT Retest	이상훈	Cancelled	0%	2026-06-20 02:37	   
8	CBT	Ishihara	Right Eye	CBT Retest	Bob	Cancelled	0%	2026-06-20 02:35	   

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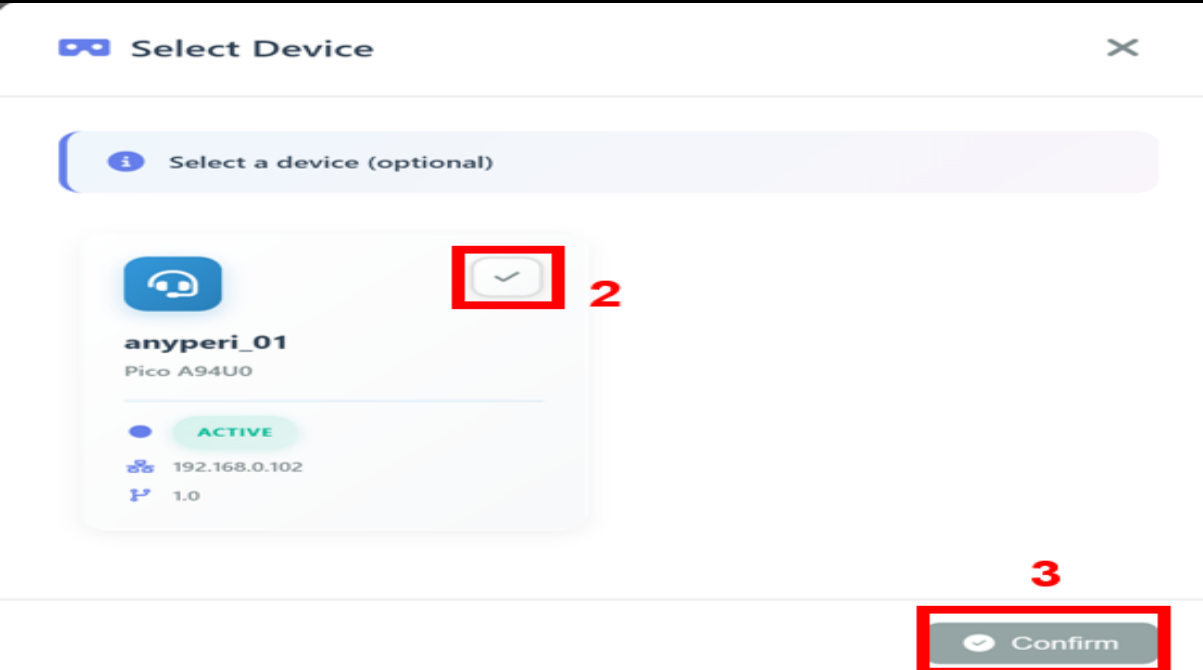
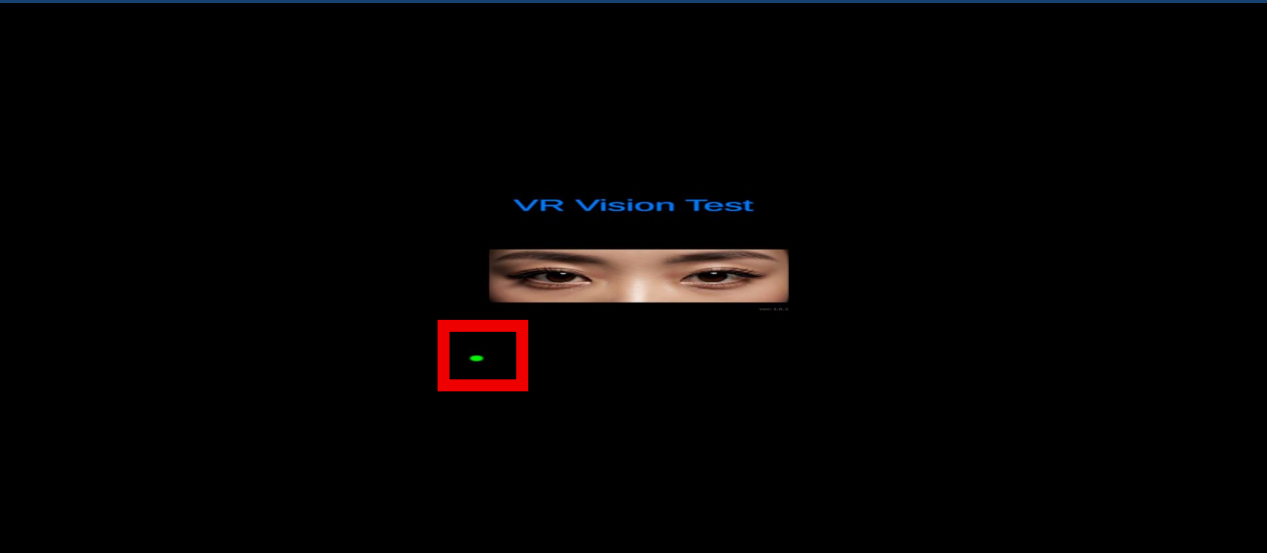
5 — Configuración del Software

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IF NO DEVICE APPEARS IN SELECT DEVICE

Why does this happen?

The headset disconnects automatically when not in use — this is by design for battery optimization.

1

Close this window

Close the empty Select Device dialog.

2

Go to the VR headset and put it on

Walk over to the headset station and wear it. If you can see the **woman's eye image** inside, the device is on and connected to the network.

3

Return to the browser and reopen Select Device

Come back and open the Select Device window again.

✓

Select the device and press Confirm Done

The headset will now appear in the list. Select it and continue.

5 — Configuración del Software

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Real-time Monitoring

Test Type

VFT-24-2

Test Name

Visual Field Test

Patient Information

Sien

Status

Pending

Select Device

anyperi_01

▶ Start Test

Test Progress

Progress

0%

Duration00:00

Completed0

Pending108

Result

Fast 24-2 (OS)

standard - medium

30°

Fast 24-2 (OD)

standard - medium

30°

Close

5 — Configuración del Software

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Real-time Monitoring

Test Type

VFT-24-2

Test Name

Visual Field Test

Patient Information

Sien

Status

In Progress

Select Device

anyperi_01

Pause

Stop Test

Test Progress

Progress

0%

Duration00:03

Completed0

Pending108

Result

Fast 24-2 (OS)

standard - medium

30°

Fast 24-2 (OD)

standard - medium

30°

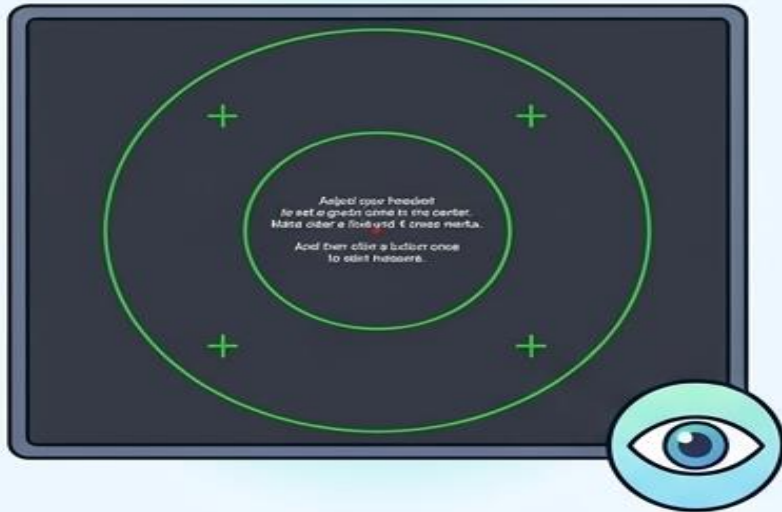
Close

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CALIBRATION & START PROCESS

1: CONFIRM CALIBRATION VIEW

Upon starting, you will see this screen. Ensure the red dot is visible and central.

2: ADJUST HEADSET FIT

Adjust the straps and lens clarity until the center circle and 4 cross marks are sharp and clear.

3: PULL TRIGGER TO BEGIN

Look at the central red dot. Pull the trigger on your controller to confirm and start measurement.

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· Alineación Visual y Enfoque

If the image is not clear after wearing the VR headset

Adjust the lenses **manually** or use the **IPD adjustment feature** in the VR to achieve a **clear image**.

Check before starting the test!

1 Adjust the lenses manually

Move the lenses left or right to the position that best matches your eye spacing.

💡 When both sides of the screen are clear and overlapped into one, that is the optimal state.

Blurred or double image

Clear and single image

2 Use the IPD adjustment feature in the VR

Use the IPD (interpupillary distance) adjustment function in the VR menu to set the value that looks clearest.

💡 Change the IPD value and find the setting that looks the clearest.

IPD too low

Optimal IPD

IPD too high

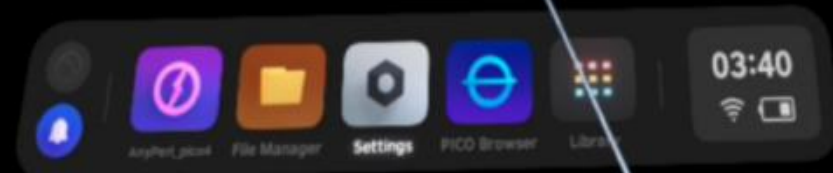
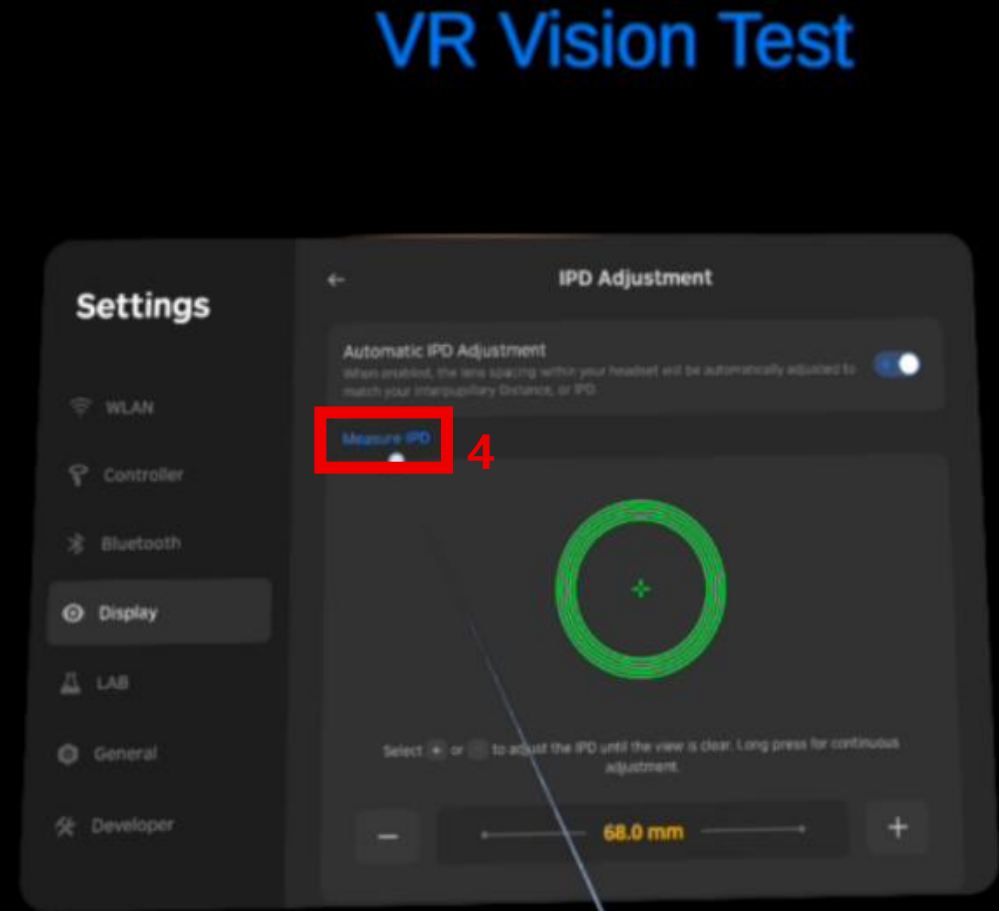
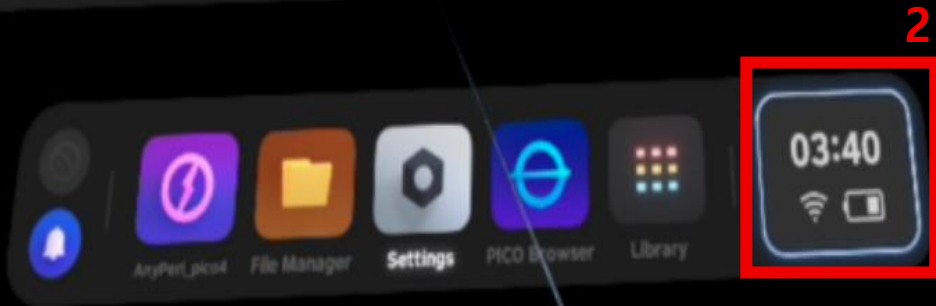
Maintaining a clear image is essential for accurate test results.

Always check the focus before starting the test.

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· Alineación Visual y Enfoque

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Automatic IPD Adjustment

PICO automatically adjusts the lens spacing in your headset to match your Interpupillary Distance, or IPD, to provide an optimal viewing experience. The cameras in your headset will capture images around your eyes, which are processed in real time on your device and converted into IPD and index values that will be used to load and adjust your lens spacing the next time you put on your headset. You can turn this off at any time. [Learn more](#)

Agree and Continue

5

Automatic IPD Adjustment

[Help](#)

Adjustment Completed

If your view is not clear enough, select [Manual Adjustment](#)

Done

6

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· Alineación Visual y Enfoque

AUTO-TRACKING SYSTEM

Auto-tracking — fixation & gaze monitoring

Auto

Before the test begins, a **fixation point** will appear at the center of your view.

Keep your gaze **steadily focused** on this point until the test starts. This allows the system to lock on to your eye position.

Maintain gaze on the center point throughout the entire test. The auto-tracking system continuously monitors your eye position to ensure accurate measurements.

If you look away from the fixation point during the test, the auto-tracking system will detect this and you will hear a **beep sound** as a warning.

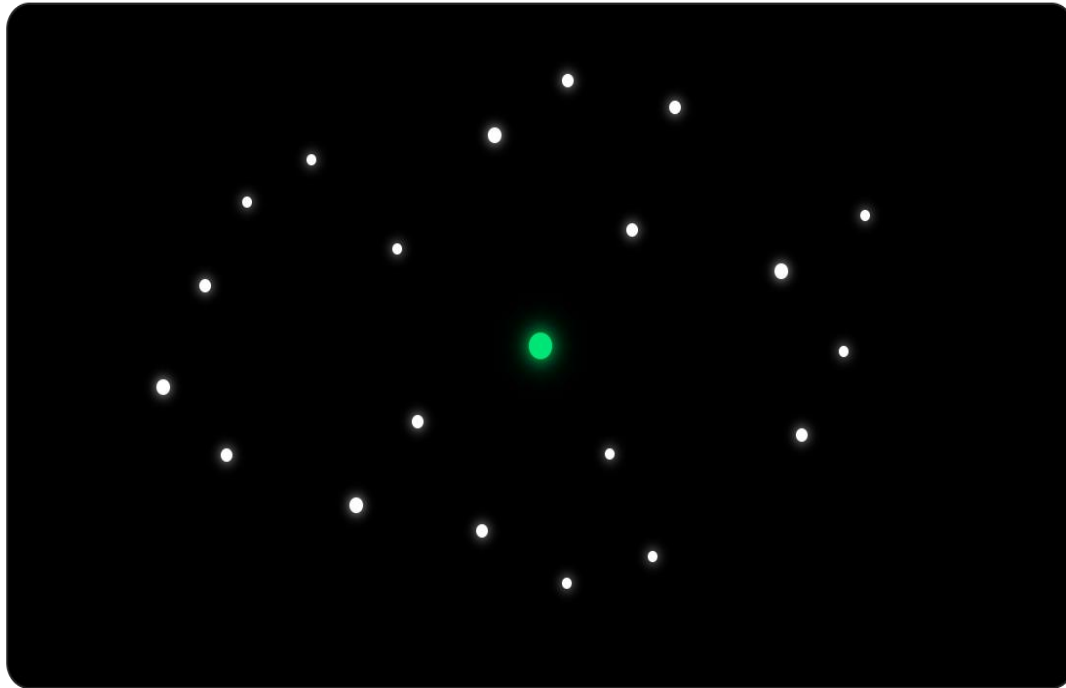
Beep heard? — Return your gaze to the center fixation point immediately to resume accurate tracking.

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VR TEST SCREEN — WHAT YOU SEE IN THE HEADSET

 Green fixation point — keep your gaze here

 White stimulus — press trigger when seen

HOW TO TAKE A MEASUREMENT

1

Wait for the test to begin

Once the test starts, the following lights will appear in sequence:

 Green light — center appears

 White light — flashes shortly after

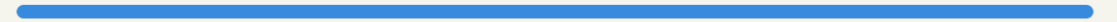
2

Press the trigger when you see the white light

Use the trigger on your **right-hand controller** each time the white light flashes.

The test consists of **54 trials**. A voice prompt will notify you of progress at intervals:

10 20 30 40 50 60 70 80 90 100%

3

If your vision feels blurry

Try **blinking** to clear your vision and continue the test normally.

Blinking briefly during the test will not affect your results. Do not rub your eyes.

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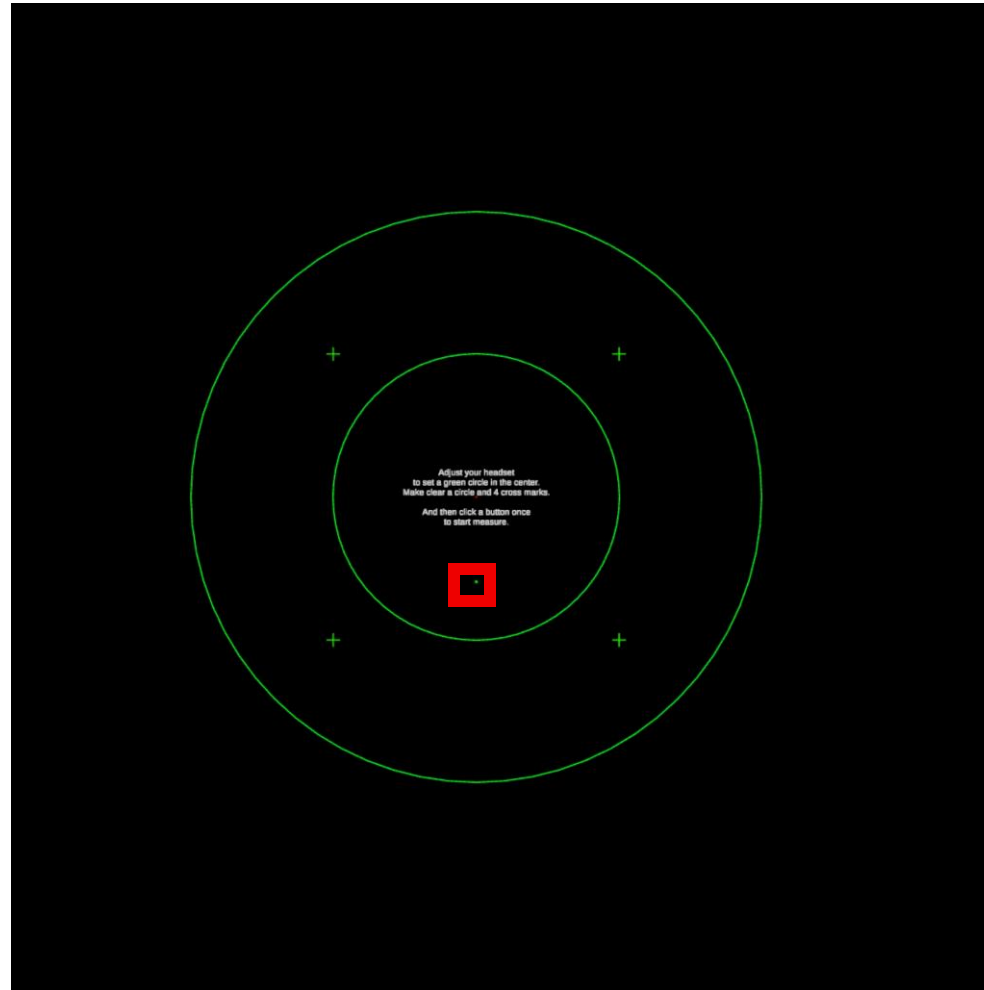
The screenshot displays the 'AnyPeri' software interface with a '검사 관리' (Test Management) sidebar. A modal window titled '새 검사 등록' (New Test Registration) is open, allowing for the configuration of a new test. The modal includes a memo field at the top, followed by a '검사 설정' (Test Settings) section. This section contains several dropdown menus: 'Test Eye' (set to '양안'), 'Test Dot Size' (set to 'III'), 'Test Pattern' (set to 'Superior-64 (Ptosis)'), 'Test Strategy' (set to 'Screening'), 'Instruction' (set to 'On'), 'Use Binocular Fixation' (set to 'Yes'), 'Foveal Threshold' (set to 'Yes'), 'Stimulus Speed' (set to 'Slow'), and 'Fixation Position Design' (set to 'Standard'). At the bottom right of the modal are buttons for '취소' (Cancel) and '새 검사 등록' (New Test Registration). The background interface shows a list of tests with columns for patient name, test name, and status, and a bottom status bar with various system indicators.

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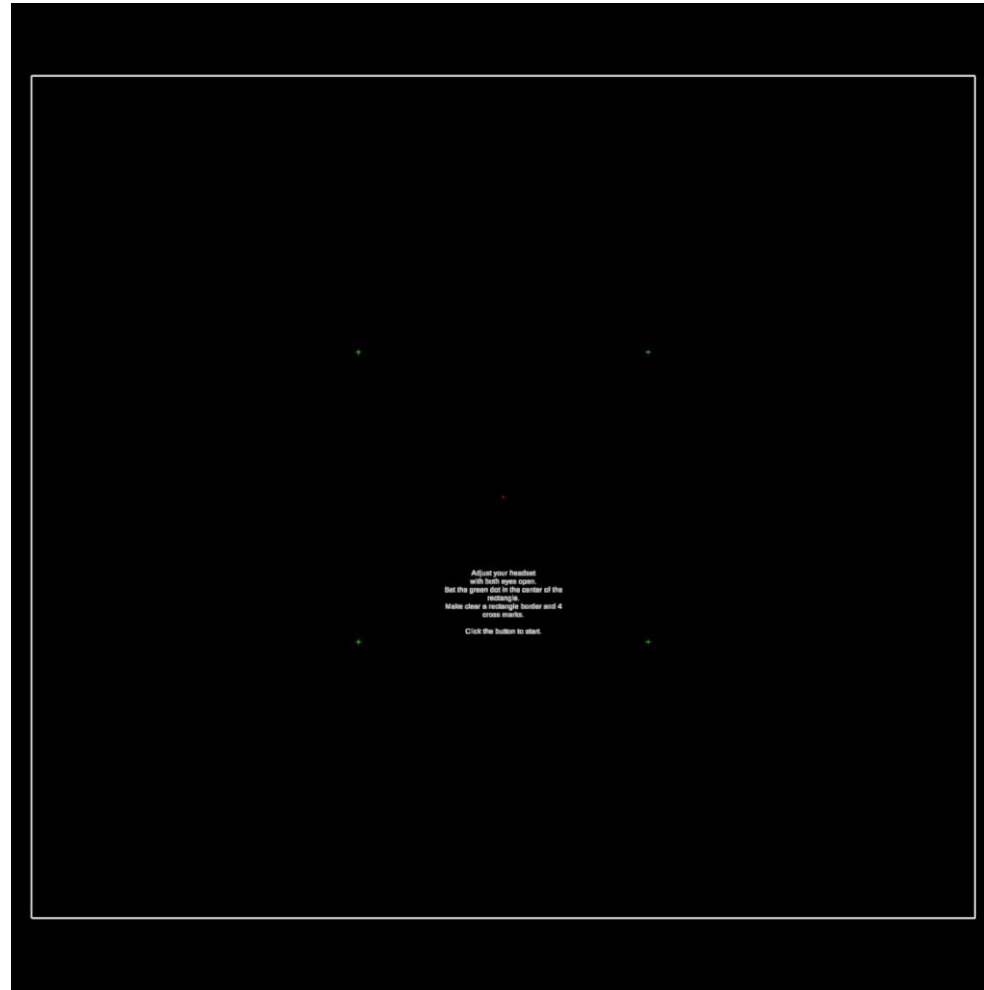
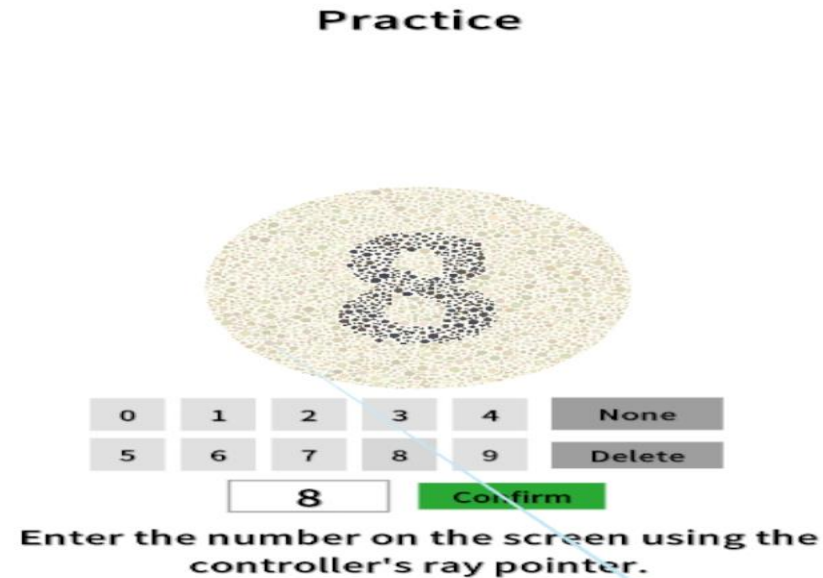
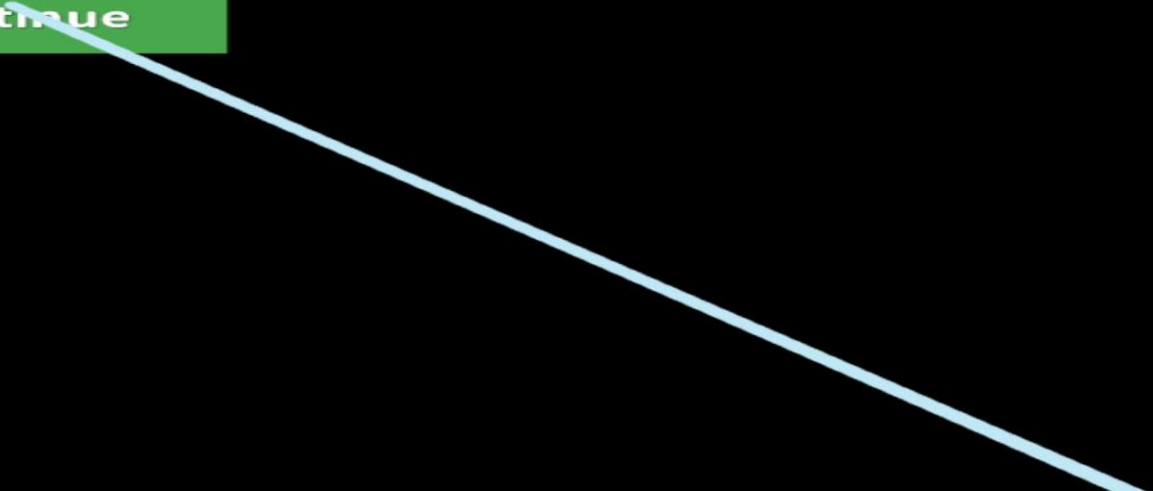
**Visual acuity test will begin.
Follow the instructions.**

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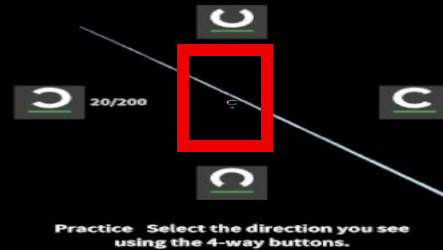
7 — Informe · Resultados y Análisis

The screenshot displays the AnyPeri web application interface. On the left is a dark sidebar with navigation options: Patient Management, Test Management, Report Management (highlighted with a red box), and Settings. The main content area is titled 'Report Management' and features a 'Patient Management' section with a search bar and a list of patient cards. The patients listed are 이상훈, 유채원, Dennis, Bob, and Alice. The 'Bob' card is highlighted with a blue border. To the right of the patient list is a detailed view for 'Bob' (P000002, Male, 55years). This view shows a grid of report cards. The first row contains three cards: 'VFT-10-2 - Visual Field Test', 'EST-120 - Visual Field Test', and 'VFT-SUP-64 - VFT-SUP-64 Retest 202604192222'. The second row contains three cards: 'VFT-24-2 - VFT-24-2 Retest 202604192214' (highlighted with a red box), 'VFT-24-2 - VFT-24-2 Retest 202604192152', and 'VFT-SUP-64 - VFT-SUP-64 Retest 202604032120'. Each card displays a PDF icon, the report title, date, time, and filename, along with download, delete, and 'Retest' buttons. The top right corner shows the user profile '홍길동' from '한국대병원'.

7 — Informe · Descargar

Report Details

VFT-24-2 - VFT-24-2 Retest 202604192214

24-2 2026-04-19 2026-04-19 22:21:26

Patient Information

PATIENT NAME

Bob

PATIENT ID

P000002

File

FILE

VFT_Report_Bob_VFT-24-2_20260419222126.pdf

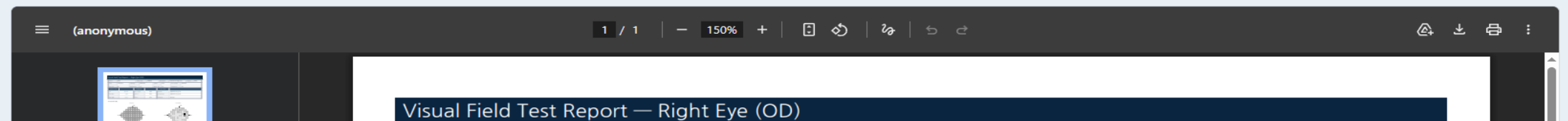
FILE

40.8 KB

Memo

Auto-generated from test ID: 618

File Preview

Close

Retest

Download

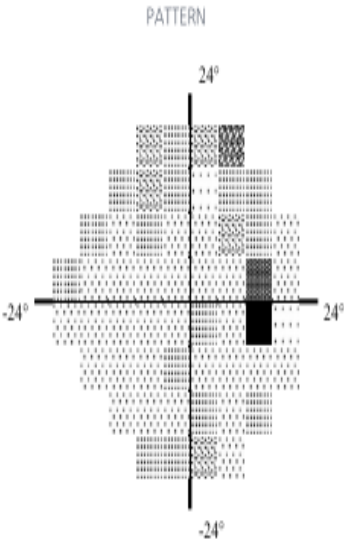
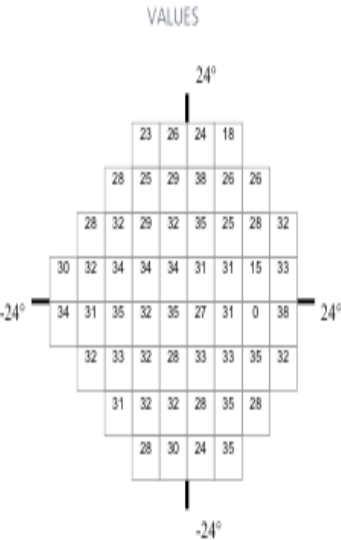
Delete

7 — Informe · Ver Detalles del Informe

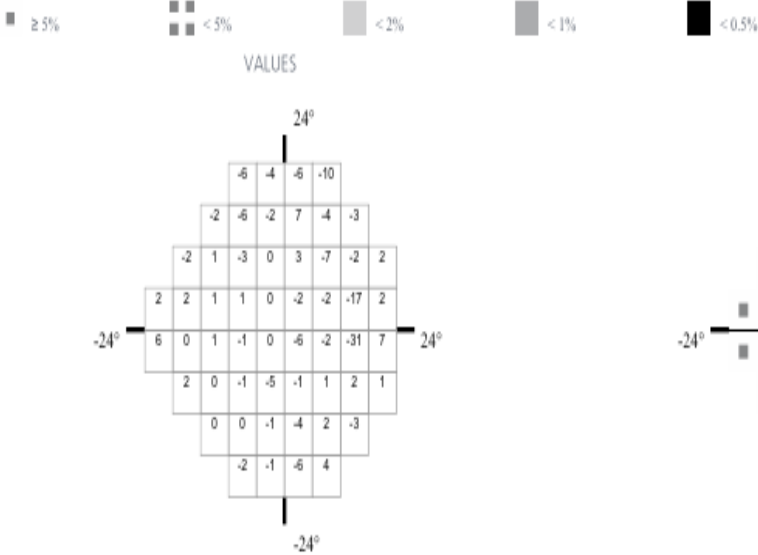
Visual Field Test Report — Right Eye (OD)

Name	Tom	Patient No.	P000003	Birth Date	1993-04-20 (Age: 33)	Gender	Male
Type	VFT-24-2	Strategy	Fast Strategy	Speed	slow	Fix. Design	standard
Dot Size	III	Instruction	on	Binoc. Fix.	yes	Foveal Thr.	yes
Global Indices		Reliability		Test Time			
VFI	96.7%	Fix. Losses	0/8	Started	2026-07-01 16:28:57		
MD (dB)	-0.93	False +	1/6	Completed	2026-07-01 16:33:24		
PSD (dB)	3.48	False -	0/4	Duration	4m 27s		

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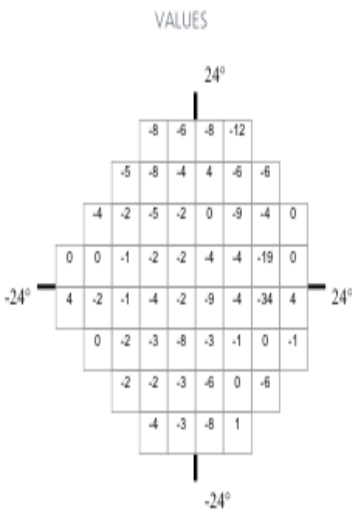
Add New Test

Test Management

NO.	TEST TYPE	STRATEGY	TEST EYE	TEST NAME	PATIENT NAME	STATUS	PROGRESS	DATE / TIME	ACTION
1	VFT-24-2	Fast Strategy	Right Eye	Visual Field Test	Tom	Cancelled	0%	2026-07-02 09:42	
2	VFT-24-2	Fast Strategy	Right Eye	Visual Field Test	Tom	Completed	100%	2026-07-01 16:33	
3	CBT	Ishihara	Both Eyes	Color Blindness Test	Tom	Cancelled	7%	2026-07-01 14:33	
4	VT	Landolt C	Both Eyes	VT Retest	Tom	Cancelled	0%	2026-07-01	

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